

INSTRUCTION GUIDE

REPLACING BEARINGS AND BUSHINGS, AND/OR CASTER FORKS

1. With flat head screwdriver, gently remove dust cover.
2. Using a 3/4" socket, remove top nylon lock nut.
3. Remove all parts and pull fork downwards.

NOTE - Be sure to remove all residue before reinstalling parts. It is strongly recommended to replace all parts at the same time. NEVER use any kind of lubricant or grease in the fork housing.

4. Reinstall parts.

NOTE - It is important to ensure that the two tension washers are installed back-to-back as shown in the illustration. Only this configuration provides the proper pressure on the fork stem.

5. Tighten nylon lock nut.
6. Replace dust cover.

